



## New Item Added for GP Information Management Software

1. Click “Settings” in the main interface of GP information management software.

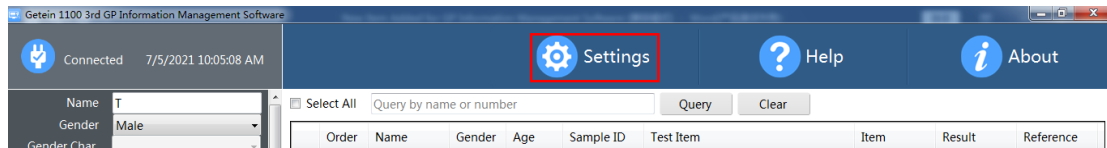

2. Select “Item” in the settings interface and input the password (gp\_poct).

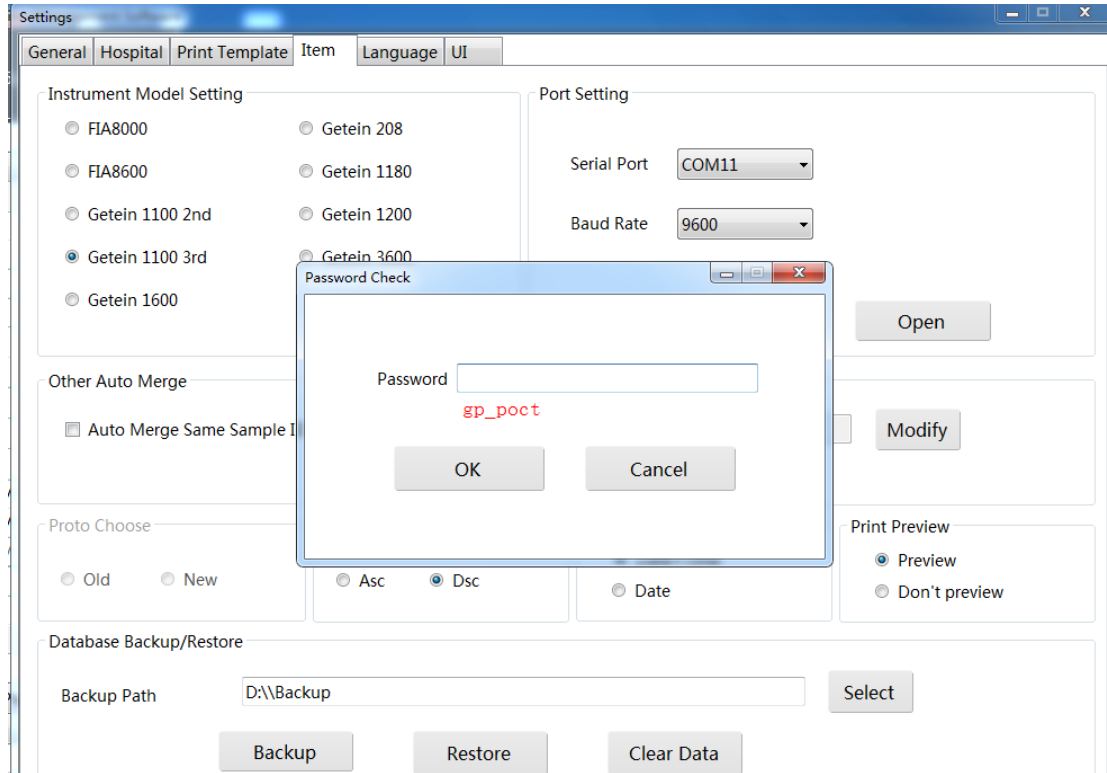

3. Input the “Neu.Ab” parameter manually as shown in the following interface.

Item No.	Item Name
1	cTnI
2	NT-proBNP
3	hs-CRP+CRP
4	cTnI/NT-proBNP
5	CK-MB/cTnI/Myo
6	D-Dimer
7	PCT
8	mAlb
9	NGAL
10	PCT/CRP
11	CysC
12	H-FABP
13	HCG+β
14	hs-CRP
15	β2-MG
16	SAA
17	IgM/IgG
18	CK-MB/cTnI/H-FABP
19	HbA1c
20	hs-cTnI
21	TSH
22	CK-MB
23	CK-MB/cTnI

Item No. 71 Test Item Neu.Ab

Sample Mode  
 Serum/Plasma  Whole Blood  Urine  Capillary Blood  Stool

Item Detail  
 Test Item Neu.Ab  
 Print Name Neu.Ab  
 Unit BAU/mL Decimal 2  
 Max 250.00 Min 0.50  
 Ref Max 1.00 Ref Min 1.00  
 Remark

Add Modify Delete Section Range

4. Then click "Add" to save the information.
5. After above operation, user can do Neu.Ab test on Getein1100 and upload the result to GP information management software.